HUMANTECHNIK

DE

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Vibrationskissen

EN

User manual

Vibrating pad

FR

Mode d'emploi

Coussin vibrant

NL

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Trilkussen

IT

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
ES

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
Cojín vibratorio


1.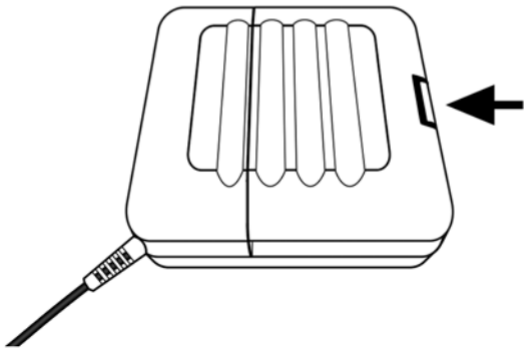


2.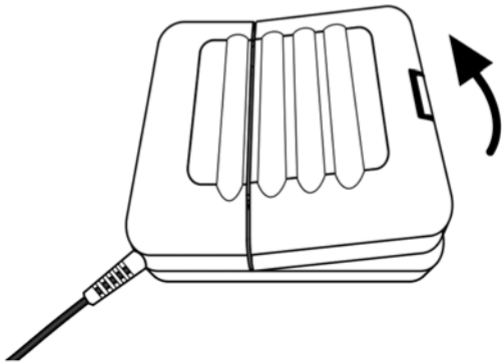


3.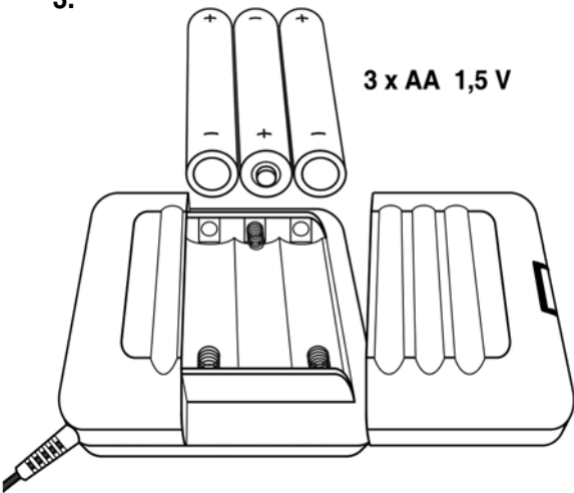


4.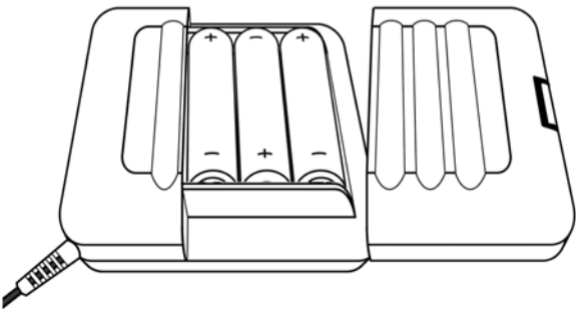

5.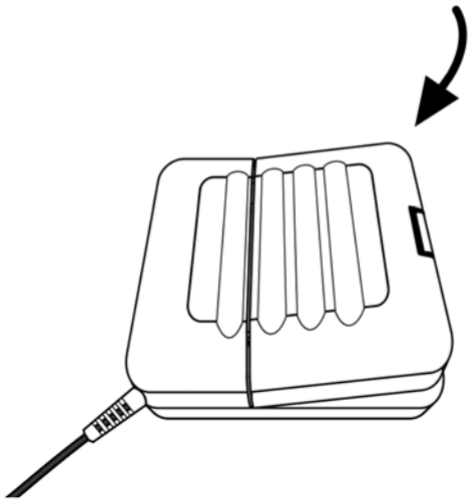

6.

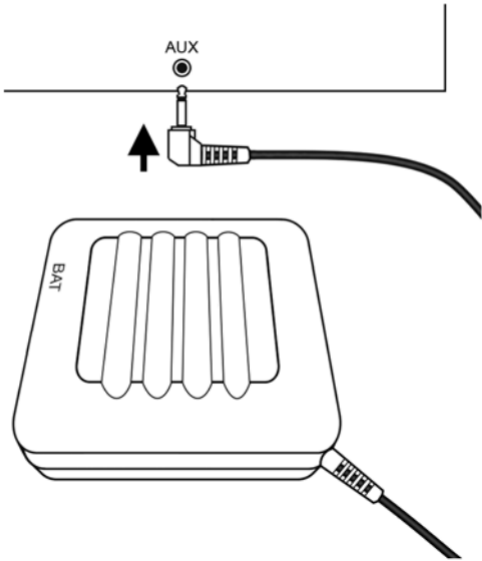Vibrationskissen

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DE

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Setup

1. Insert batteries: See pages 1 to 5.

The device vibrates twice to confirm its function.

2. Connect the plug with the AUX-socket of a *lisa-* or *signolux*-receiver (page 6).

Status indication „Low battery“

Depleted batteries are indicated by the following signs:

- The status-LED „BAT“ blinks every 4 seconds
- When activating the pad, vibration is as follows:


Please replace the batteries as soon as possible!

Using rechargeable batteries

The vibrating pad can also be powered by rechargeable batteries of size AA.

The different voltages of normal batteries and rechargeable ones make the following adjustment necessary:

1. Open the battery compartment
2. Remove the batteries
3. Underneath the front contact spring is an opening with a small adjuster switch, as depicted on the next page:

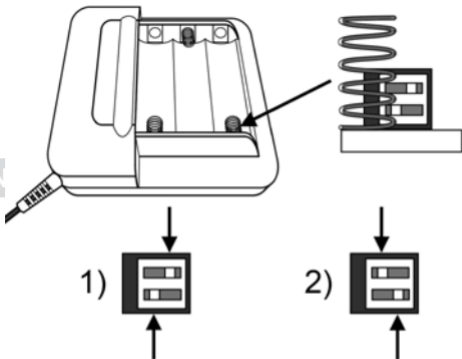Position 1): When using regular batteries

Position 2): When using rechargeable batteries

With the help of a pointed object, change the position of the small switches accordingly.

Please note: Depleted rechargeable batteries need to be recharged in an external battery charger.

Should regular batteries be used again, please re-set the adjustment switch accordingly.

EN

HUMANTECHNIK

Designed in Europe

Made in China

This device complies to the following EC directives:

- 2011/65/EG RoHS-directive
- 2002/96/EG WEEE-directive
- 1999/5/EG R&TTE-directive
- Low Voltage Directive 2006 / 95 / EEC
- Electromagnetic Compatibility 2004 / 108 / ECC

The complete declaration of conformity is available at
www.humantechnik.com

This product must not be handled as ordinary household waste!
Please dispose of used batteries according to your local regulations!

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Humantechnik GHL AG
Sarabec Ltd.