*lisa*  
RF time flash

D

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Seite 2

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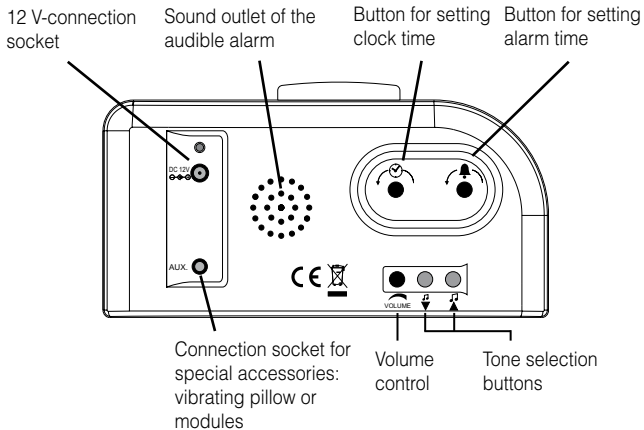
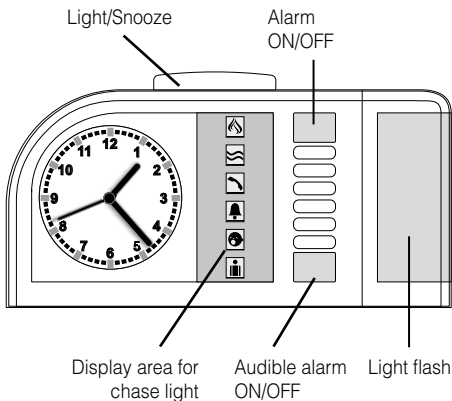
We would like to congratulate you on the purchase of our »*lisa* RF time flash« alarm clock. You have chosen a modern and reliable system. Please read the operating instruction carefully to be able to start the unit correctly and to familiarize yourself with all of the system's features.

### **Standard components**

Please check if all following components are included:

- »*lisa* RF time flash«
- 12V power supply unit
- Operating instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### **Functioning principle**

The »*lisa* RF time flash« is an electronic clock with an analogue time display and a receiver for the radio signals from various *lisa* radio transmitters. When a radio signal is detected, you are alerted in the form of a flashing light/acoustic alarm and 6 light-emitting diodes.

The following special accessories can be connected to the alarm clock:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module

### **Startup**

Plug in the power supply unit of the »*lisa* RF time flash« into a 230 V power outlet and connect the cable with the 12 V connection socket.

### **Time setting**

Turn the button for setting the time  in the direction of the arrow until the desired time is set.

### **Alarm setting**

Turn the button for setting the alarm  in the direction of the arrow until the required alarm time is set.

### **Switching on the alarm**

Switch on the alarm using the ON/OFF switch.

Switch not pressed: Alarm switched on (green outline visible around switch).

Switch pressed: alarm switched off.

### **Switching on the audible alarm**

Switch on the audible alarm using the audible alarm ON/OFF switch.

Switch not pressed: Audible alarm switched on (green outline visible around switch).

Switch pressed: Audible alarm switched off.

### Select alarm tone/melody

You can select the alarm tone/melody with the tone selection buttons. You can choose from 10 different types of tone sequences.

### Snooze Alarm

Press the Light/Snooze button to suspend an alarm which is in progress. The alarm is triggered again after about 5 minutes.

### Switching on the clock face illumination

Press the Light/Snooze button to switch on the clock face illumination for about 6 seconds.

### Setting the correct radio channel

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels. When supplied from the factory, the »lisa RF time flash« is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Set the channel you want by adjusting the channel selector switch on the underside of the unit to the required channel number. Use a small screwdriver to adjust this switch. All transmitters and receivers within your system must be set to the same channel.

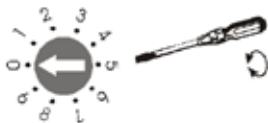
Fig. setting the correct radio channel.


## Transmission Signal Recognition

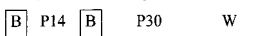The »lisa RF time flash« converts the various transmission signals into varying sequences of flash and signal displays.

### Telephone:

Telephone LED lights


Flashing-  
sequence

### Doorbell I:

Türklingel-LED leuchtet

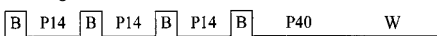
Flashing-  
sequence

### Doorbell II:

Doorbell LED flashes

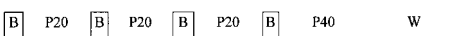Flashing-  
sequence

### Babymonitor:

Baby LED lights

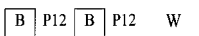
Flashing-  
sequence

### Pager:

Pager LED lights


Flashing-  
sequence

### Alarm:

All LEDs flash

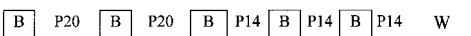
Flashing-  
sequence

### Water alarm:

Water Alarm LED lights

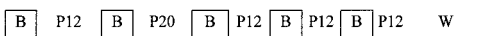
Flashing-  
sequence

### Fire alarm:

Fire Alarm LED lights

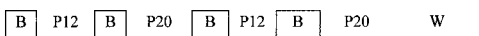Flashing-  
sequence

**P12** Pause 1,2 seconds

**P30** Pause 3,0 seconds

**P14** Pause 1,4 seconds

**P35** Pause 3,5 seconds

**P20** Pause 2,0 seconds

**P40** Pause 4,0 seconds

**B** Flash

**W** Repeat

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any flash alarm caused by *lisa* signals. The indicator LEDs light the normal time (40 seconds).

### Recalling the most recent signal

The »lisa RF time flash« stores each signal it receives for a period of 3 minutes.

During this time, you can call up the signal again by pressing the Light/Snooze lisa signal cancel switch button. The signal is then displayed on the indicator LEDs. A flash signal is not triggered.

### Radio range

The lisa transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there maybe many other sources of radio waves which can disrupt the original signal.

- Having units operating at similar frequencies located close to one another may also result in mutual interference.

### Special aspects

- The alarm signal is repeated after 5 minutes (autosnooze) unless you press the Light/Snooze button or the alarm ON/OFF button.
- The quartz mechanism of the alarm clock has a backup power supply. The clock will continue to operate for about 2 hours if there is a power failure. The alarm function and the illumination are deactivated during this period.
- The clock face illumination is also activated when the alarm is triggered.
- In addition to the green marking of the Alarm ON/OFF switch, the snooze button is also illuminated when the alarm is triggered.
- The »lisa RF time flash« uses a chase light for signalling an alarm. During an alarm the 6 LEDs at the frontside will be switched alternately, creating the chase light.
- The »lisa RF time flash« converts the various transmission signals (Telephone, Doorbell I+II, Baby-monitor, Pager, Alarm, Water alarm, Fire alarm) into varying sequences of flash and signal displays.

### **Maintenance and care**

The »lisa RF time flash« does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### **Warranty**

The »lisa RF time flash« is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment

and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

### Technical data

Power supply: 230 V, 50 Hz; 12V DC 1000mA

Power consumption: approx. 2 W

Height: 86 mm

Width: 175 mm

Depth: 53 mm

Weight: 337 g

This device satisfies the following EU directives:

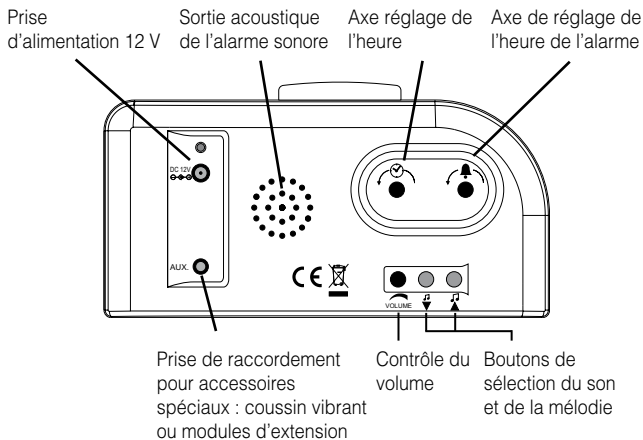
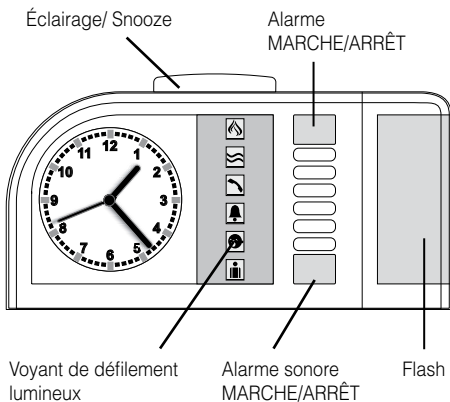
- 2002/95/EG RoHS directive
- 2004/108/EG EMC directive
- 2002/96/EG WEEE directive
- 2006/95/EG Low voltage directive
- 1999/5/EG R&TTE directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humantechnik.com](http://www.humantechnik.com).

Technical specifications subject to change without notice.

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**Humantechnik GmbH**Im Wörth 25
**Humantechnik GHL AG**Rastatterstrasse 9
**Sarabec Ltd**15 High Force Road