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
### Filterelement/Filter element bachmaiER b30

Frequenz in Hz	Mittelwert der Schalldämmung in dB	Standardabweichung in dB	Angenommene Schutzwirkung APV in dB
Frequency Hz	Average noise reduction in dB	Standard deviation in dB	Assumed protection (APV) in dB
63	27,5	3,8	23,7
125	29,4	4,8	24,6
250	28,4	5,4	23,0
500	28,3	6,4	21,9
1000	28,1	5,7	22,4
2000	31,2	4,6	26,6
4000	25,7	4,5	21,2
8000	40,8	5,7	35,1

Filter	bachmaiER b10	bachmaiER b15	bachmaiER b25	bachmaiER b30
SNR-Wert in dB	11	15	23	25
H-Wert in dB	10	13	21	24
M-Wert in dB	9	14	21	23
L-Wert in dB	10	14	21	23

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We are pleased that you have done something good for your hearing. Your bachmaier hearing protectors have been matched to your individual requirements. We would now like to give you a few valuable tips so that you can enjoy the benefits of this quality product for a long time.

## Correct use of a bachmaier

Take the custom mould between thumb and forefinger at the gripper strip and twist it, using light pressure, into the auditory canal until it sits comfortably.

## The right way to use a bachmaier with a handle

Hold the custom made ear plug with your thumb and index finger and apply gentle pressure as you twist it into the auditory canal until it is sitting comfortably. Incl. attachment point for cord.

## Storage and care

Store your hearing protectors in the case supplied. Clean the bachmaier b10·b15·b25·b30 after each use. Remove any ear wax residue using the cerumen stick. With them the filters can be easily removed from the earmoulds. We recommend the bachmaier **oto-fresh®** products for the optimum cleaning and care of your hearing protectors. You can obtain them from your specialist.

**IMPORTANT:** The attenuation filter inside the mould must not become wet!

## Usage Instructions

If the instructions for use are not observed, the protective effect of the bachmaier b10·b15·b25·b30 can be significantly impaired. The bachmaier b10·b15·b25·b30 achieve their protective effect only if you handle them carefully and wear them continuously in noisy environments. If you remove them in noisy environments from your ears, even for a short time, you can damage your hearing. After about 3 years, changes in the ear canal may change the attenuation values; bachmaier recommend having a new earmould made after this period.

**WARNING:** The ear plugs should not be used when there is a danger that the connecting cord could get snagged or caught in moving machinery. In this case we recommend removing the cord.

### 3 products with 1 earmould!

The earmould is designed in such a way that by removing the filter and inserting the b.IEM ER headset or electronic shooter, you get high quality, custom made headphones or level-dependent, active hearing protection. Ask your audiologist – he'll be delighted to advise you!

bachmaier b10·b15·b25·b30 is a custom-made comfortable hearing and has passed the type test to DIN EN 352-2 tested at PZT GmbH in 26389 Wilhelmshaven/Germany, An der Junkerei 48F, under Test No. 771402-1 on 11.07.2014.

b.IEM ER are high quality earphones with type examination to DIN EN 352-2 and 352-6 and were tested at PZT GmbH in 26389 Wilhelmshaven/Germany, An der Junkerei 48F, on 07.02.2011 under Test No. 771006-1.

bachmaier shooter provides active, custom made hearing protection with type examination to DIN EN 352-7. Tested at PZT GmbH, 26389 Wilhelmshaven/Germany, An der Junkerei 48F, on 30.08.2012 under Test No. 771203-2.

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Art.-Nr. 8270/11,14 Rev. 5 · ©bachmaier