MA-1

D

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Seite 2

Akustikmodul MA-1

GB

Operating Instructions

Page 6

Acoustic Module MA-1

F

Mode d'emploi

Page 10

Module acoustique MA-1

NL

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Pagina 14

Akoestische module MA-1

I

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Pagina 18



Modulo acustico MA-1

E

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Página 22

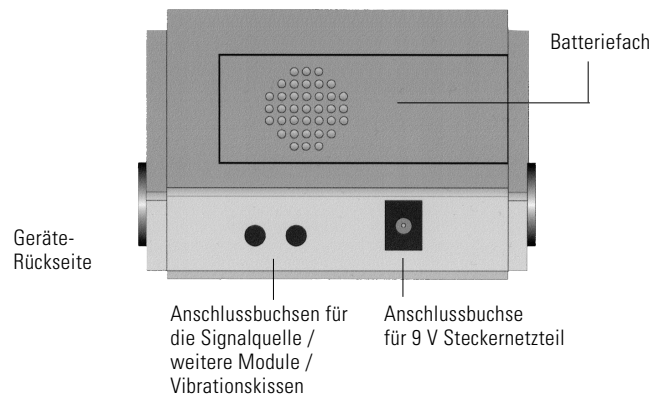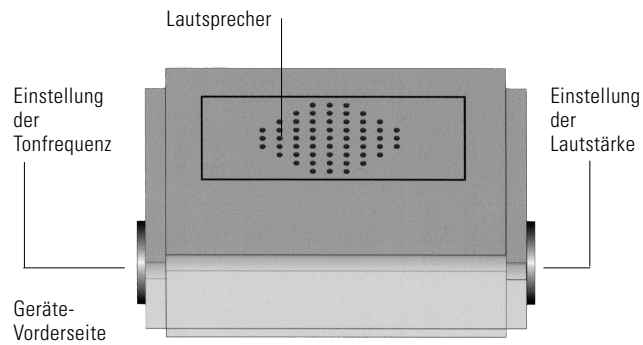Módulo acústico MA-1

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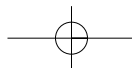
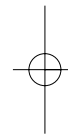
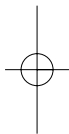
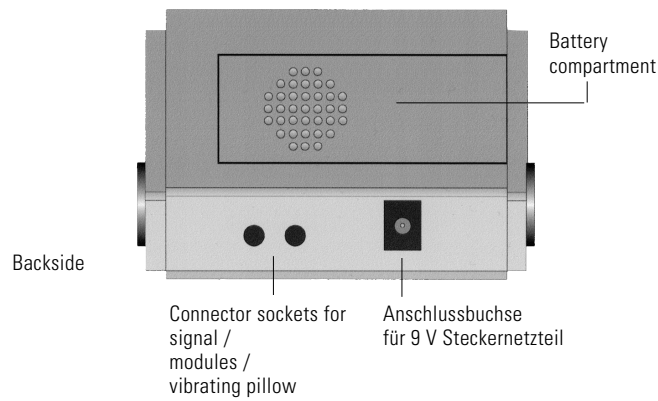
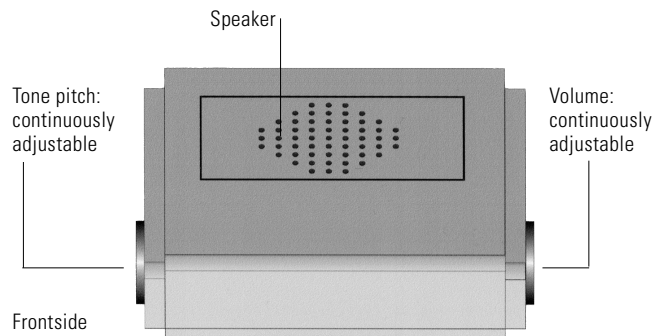
With the *Acoustic Module MA-1* you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a very loud acoustic tone.

Congratulations on purchasing the *Acoustic Module MA-1*. You have selected a modern and reliable system. With the Acoustic Module MA-1 you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a very loud acoustic tone.

Standard components

Please check if all listed items are included:

- *Acoustic Module MA-1*
- Connecting cable
- 9 V-power supply unit
- Warranty card

Installation

Attach the connecting cable to one of the two connector sockets for the signal source at the back of the *Acoustic Module MA-1*. Plug the other end of the cable into a unit with a connector socket for a vibrating pillow, e.g. digital clock DS-1, vibrating clocks VC-1 / VC-10, time *flash / lisa* time *flash*, *lisa* receiver (table flash lamp, radio flash lamp, portable radio receiver). Plug the 9 V-power supply unit in the respective socket at the back of the *Acoustic Module MA-1*.

Note:

A vibrating pillow or another module (e.g. Flash Module) can be plugged into the empty connector socket at the back of the *Acoustic Module MA-1*.

Signalling

Upcoming signals (e.g. wake-up alarm, *lisa* alarm) are signalled by a very loud acoustic tone.

Special features

Volume: continuously adjustable (max. 100 dB)

Tone pitch: continuously adjustable (100-2000 Hz)

Power failure backup:
with a 9 V battery
Battery operation (not included)

The *acoustic module MA-1* can also be operated on just a 9V battery

Maintenance and care

The *Acoustic Module MA-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

Warranty

The *Acoustic Module MA-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

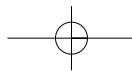
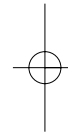
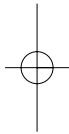
Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

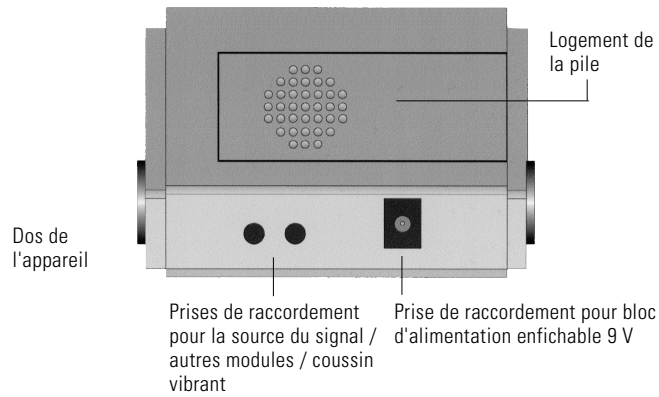
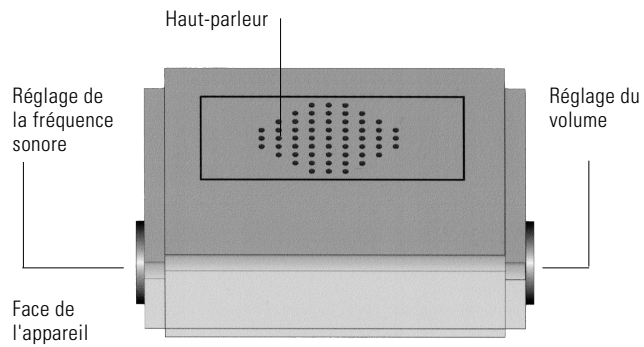
Always specify the product number in any event.

Technical Data

Power supply: 9 V-power supply unit at 230 V, 50 Hz
or with a 9 V battery

Tone pitch: 100-2000 Hz continuously adjustable

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Humantechnik GHL AG