## Sicherheitshinweise / Safety regulations:

Contiene / Indeholder / Enthält / Περιέχει /  
Contains / Contient / Contiene / Bevat /  
Contém / Sisältää / Innehåller: Cyanacrylat

### R 36 / 37 / 38

ES: Irrita los ojos, la piel y las vías respiratorias.  
DA: Irriterer øjnene, åndedrætsorganerne og huden.  
DE: Reizt die Augen, Atmungsorgane und die Haut.  
EL: Ερεθίζει τα μάτια, το αναπνευστικό σύστημα και το δέρμα.  
EN: Irritating to eyes, respiratory system and skin.  
FR: Irritant pour les yeux, les voies respiratoires et la peau.  
IT: Irritante per gli occhi, le vie respiratorie e la pelle.  
NL: Irriterend voor de ogen, de ademhalingswegen en de huid.  
PT: Irritante para os olhos, vias respiratórias e pele.  
FI: Ärsyttää silmiä, hengityselimiä ja ihoa.  
SV: Irriterar ögonen, andningsorganen och huden.

### S 23–24 / 25–26

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## General product information

Unigluе is a rapid-hardening, solvent-free single component adhesive. This instant adhesive is used to bond different kinds of earmold materials. Please note that the surfaces of silicone earmolds should be pre-treated with Primer (REF 39700) before gluing sound tubes.

**WARNING: Cyanoacrylate: Danger – Bonds skin and eyelids in seconds. Keep out of reach of children!**

### Storage:

Unigluе should always be stored in a dry and cool place (at 2–8 °C / 35.6–46.4 °F in the refrigerator). Always keep container closed. Storable 21 months from the date of production when stored properly in the original container (the date of production can be derived from the first four digits of the batch number: e.g. 3010 = week 30, year 2010).

### Usage:

1. Close container properly after usage.
2. Avoid contact with skin. Before handling, protect hands with appropriate gloves.

### Handling:

Apply Unigluе always very sparingly. The thinner the bonded joint, the firmer the bond and the more quickly hardening takes place. The ultimate strength of the bond is achieved within 12 hours.

The usage of Unigluе requires handling by qualified personnel considering the relevant safety data sheet. Working with Unigluе should be done under a vent. Please see safety data sheet of this product for disposal.

### Handling instruction:

#### 1. Preparation

Best results are achieved on clean surfaces. Remove dust and harder particles by using compressed air. It is advisable to clean the surface beforehand with Surface Cleaner (REF 3160ff) from greasy and oily remainders. For the generation of compressed air we offer various compressors and our *Cedis* AirPower spray.

#### 2. Gluing

Apply a thin film of Unigluе. Depending on the material and the amount of glue used, press the parts to be glued together tightly. Due to the capillary action the adhesive penetrates the smallest gaps.

Advice: For the application of the Unigluе instant adhesive we offer dosing tips in the sizes 0.5 mm (REF 39505) and 1.0 mm (REF 39504).