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**oto-fresh® dispenser**, 30 tissues,  
pleasant freshness with just one wipe ...

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**oto-fresh® drying and cleansing beaker**, one beaker for  
both, cleansing and disinfecting with tablets or drying with  
dehumidifying capsules

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**oto-fresh® cream**, for a fresh and pleasant feel

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
**oto-fresh® hearing protection starter kit**,  
freshness and safety for in-between times and when  
you're on the go

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*Music enjoyment and  
communication*

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*We are delighted that you have chosen the tried and tested sound experience for bachmaiER ear protection and any ER all over the world. To allow you to benefit from the advantages of this high quality product for as long as possible, we would like to give you a few valuable tips.*

## Correct use

*To use your bachmaiER 9-15-25 hearing protection as headphones, remove the filter from the earmould by applying gentle pressure. The filter and headphones have the same diameter, making them easy to swap. Filters out – headphones in! Switch on/off or take a phone call by pressing the button. The built-in microphone makes it easy to make phone calls, even when you don't have your hands free.*

## Storage and care

*When not in use, please store your headphones in the case provided. Clean the earmould using a cleansing cloth from the bachmaier oto-fresh® range of cleansing and care products. These are available from your hearing specialist. **IMPORTANT: Do not allow the acoustic filter in the earmould to get wet!***

## User information

*Enjoy your music at a moderate volume and avoid listening for long periods of time or at too loud a volume, as this can damage your hearing.*

*In some instances you may experience a disturbing noise when using your headphones. This is not a defect! The input/output impedance may not be compatible with some products. To achieve perfect sound quality we recommend using an adapter available from any specialist retailer if one is not included as standard with your equipment. Please do not hesitate to contact us should you have any other queries.*

## bachmaiER

*The hearing protection earmould for communication and listening enjoyment. The individually manufactured bachmaiER hearing protection combines the advantages of comfort with safety, uniform sound insulation. In accordance with EN 352-2.*

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Art-Nr. 8251.08.12 Rev. 6 © bachmaier